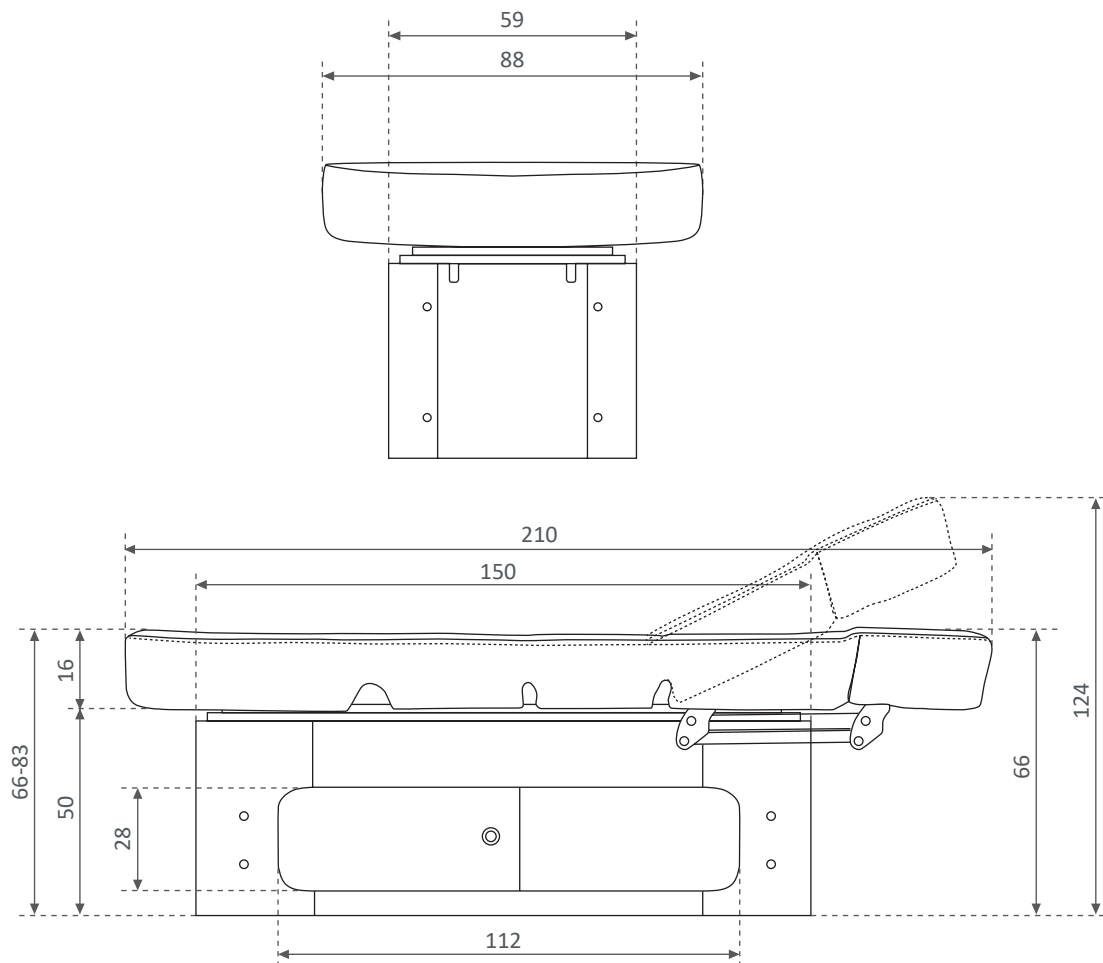
# Camilla cromoterapia Boreal

ref: E-1361

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### How to fill the bags with water.

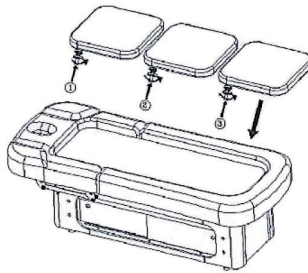
1. Loose the threaded holes and fill 22kg cool water into each water adn the tighten.
2. Put the water bags parallel on the bed surface.

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### How to change water.

1. Take away the bags from the bed carefully. Then remove the bags to the washroom and open the threaded holes to pour out the water.
2. Pour the water purifying agent in the bags firstly. Then fill the bags with water and tighten.

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

### How to remove the water bags.

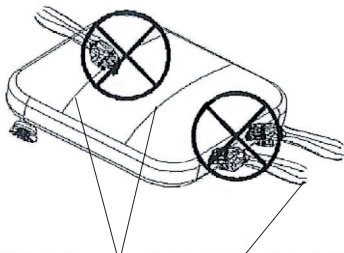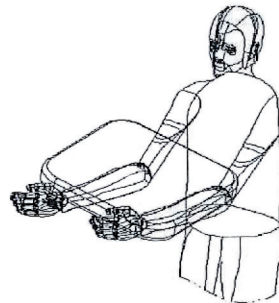Always ensure to remove the bags with two hands as the below sketch showing. So they can not be broken.

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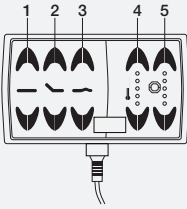
Do not pull the stitching around the bags, in order to keep the bags well.

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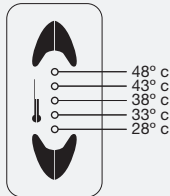
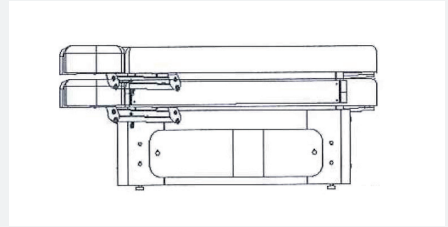
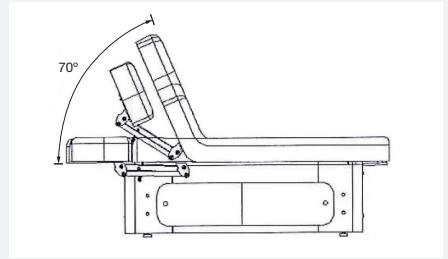
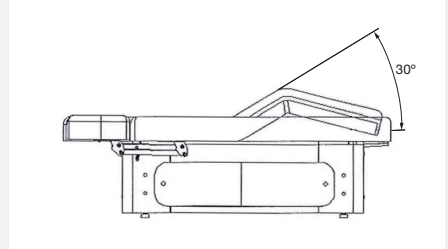
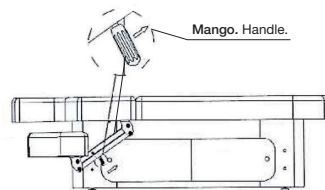
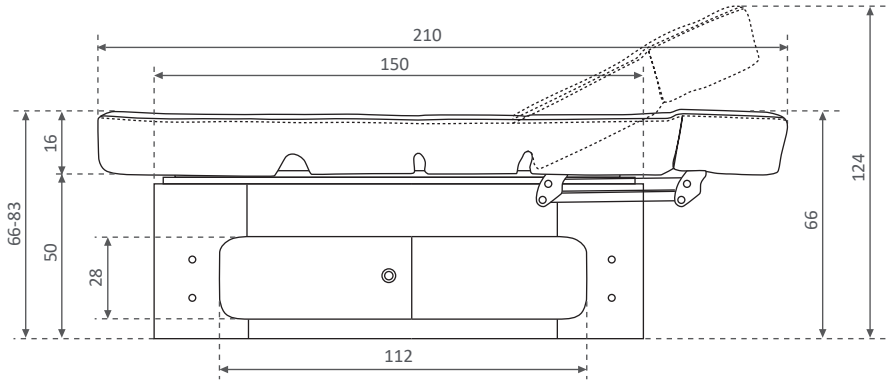
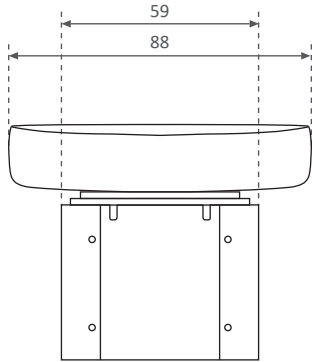
Move the handle to adjust the armrests as the arrow direction.

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## MAINTENANCES

### 1. Daily cleaning.

You can wipe out the dust by using soft dry towel.

### 2. Eliminating the heavy dirt.

As we all know, it is hard to eliminate the heavy dirt by using normal dry towel. In this case, you can use soft towel with diluted neutral detergent to clean it first. And then you can use dry towel to scrub.

### 3. It's prohibited to use benzene, diluent, gasoline or other solvent.

These solvents may damage the leather or it may cause the corrosion of the products' appearance.

### 4. Storage and operating environment.

Don't put the product under the strong sunlight and moisture environment. Otherwise it may cause the damages of the product.

5. It is allowed to use this product under the temperature from -5°C to 50°C and the humidity from 45% to 85%.

ITALORCA, S.L.  
Pol. Ind. Saprelorca. Buzón 72  
Avda. Río Guadalentín, D14  
30817 Lorca (Murcia)

Tlf. Sede Central: 968 476 411  
Mail: [italor@italor-spain.com](mailto:italor@italor-spain.com)

Contacto comercial  
Tlf.: 636 398 099  
Mail: [comercial@italor-spain.com](mailto:comercial@italor-spain.com)

Atención Técnica al Cliente  
Tlf.: 671 685 758  
Mail: [sat@italor-spain.com](mailto:sat@italor-spain.com)

[www.italor-spain.com](http://www.italor-spain.com)